Instructions for Use

NO BUBBLE TURP

Irrigation Set

Wismed CC
71 Princess Alice Avenue,
Durban 4001, South Africa
Tel: +27 31 206 0177
orders@wismed.co.za

TNMC Devices (Llc)
Ulica Franje Krežme 1
10000 Zagreb, Croatia
Tel: +38 59 1622 9658
tnmchr@tnmc.biz

Instructions for Use (EN.)	2
Mode d'emploi (FR.)	5
Gebrauchsanweisung (DE.)	8
Istruzioni per l'uso (IT.)	11
Gebruiksaanwijzing (NL.)	14
Bruksanvisning (SE.)	17
Instrukcja Używania (PL.)	20
Instruções de utilização (ES.)	23
用説明書 (JP.)	26
Vejledning til brug (DAN.)	29
Instruções de utilização (PT.)	32
Kullanım Talimatı (TR)	35
Uputstva za upotrebu (HR.)	38
Symbols	41

NO BUBBLE TURP

Instructions for Use

EN

CAUTION

WS-IFU-005 TURP V01

PAGE 02

INDICATIONS

For use with urological and /or endoscopic surgical procedures requiring irrigation. Only sterile irrigation fluids should be used during urological and/or surgical procedures requiring irrigation.

CONTRAINDICATIONS

None known

USERS

Physicians and other qualified medical personnel.

PATIENT TARGET GROUP

Adult patients requiring diagnostic and/or therapeutic cystoscopy including Transurethral Resection of Prostate.

PREPARATION FOR USE

Inspect the package for any mechanical damage.

Do not use if package is damaged

CAUTION

All necessary manipulations during the preparation and during the intervention must be carried out under sterile conditions.

The No Bubble TURP Irrigation set should be used only by physicians and other qualified medical personnel who are familiar with equipment normally used in endoscopic procedures.

POTENTIAL COMPLICATIONS

Any irrigation product may cause the following adverse effects:

- Excessive distension of urinary tract
- Extravasation of irrigating fluid

The patient should be routinely monitored in accordance with the accepted standards for endoscopic procedures.

INTENDED USE AND DISPOSAL


The No Bubble TURP Irrigation set is intended to be used on an individual patient during a single urological procedure and for continuous bladder irrigation for up to 24 hours after the procedure; and then disposed of in accordance with the hospital medical waste handling procedure. It is not intended to be reprocessed and used again. If the device is reused the patient would be at risk of infection and other complications.

DEVICE DESCRIPTION

The No Bubble TURP set is a sterile, single use irrigation device for conducting fluids from its source (bag or bottle) into the patient's lower urinary system. It is an externally communicating device and requires an interphase typically in the form of a cystoscope to which it connects.


The No Bubble TURP set is equipped with soft drip chambers and water dispersers to reduce the number of bubbles entering the set during rapid irrigation. The colour coded pinch clamps are placed in sterile field during the procedure to allow the scrub nurse or surgeon to change from one irrigation bag to another without interrupting the procedure.

INTRA-OPERATIVE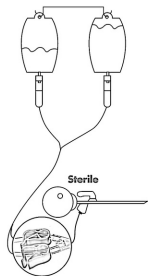

The clamps are located in sterile field. The switch from empty to full bag can be done easily by the scrub nurse or the surgeon.

POST-OPERATIVE

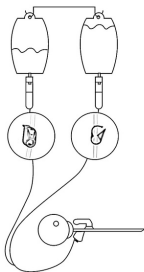The double barrel tubing can be separated easily after the procedure and the clamps moved under the drip chamber to facilitate post-operative irrigation.

NO BUBBLE TURP SET

Complete No Bubble TURP irrigation set

INSTRUCTIONS FOR USE


WS-IFU-005 TURP V01

PAGE 04

1. Peel apart the pouch to open and remove the product using the sterile technique.

2. Close both pinch clamps in sterile field.

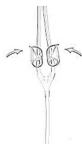3. Remove the protective spike covers and insert both spikes into the irrigation bags.


4. Squeeze both drip chambers in order to fill them with fluid (approx. 3/4 full)

5. Release one pinch clamps to remove air in the tube, then close. Repeat with the other pinch clamp to complete priming.

6. Attach the silicone tube to the patient interphase / resectoscope.

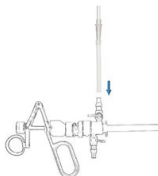7. For continuous bladder irrigation, detach the silicone tube and connect the step-down connector to the catheter.

8. Use the roller clamp to adjust the flow rate.

9. Pull the tubes from each other to separate them.

10. Move one pinch clamp up the tube so that it is directly under the drip chamber.

11. Then close the clamp and move the second clamp up.

12. The two pinch clamps should never be open at the same time.

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AB Yetkili Temsilcisi