Instructions for Use

SMART BULB

Irrigation System

Wismed CC
71 Princess Alice Avenue,
Durban 4001, South Africa
Tel: +27 31 206 0177
orders@wismed.co.za

TNMC Devices (Llc)
Ulica Franje Krežme1
10000 Zagreb, Croatia
Tel: +38 59 1622 9658
tnmchr@tnmc.biz

Instructions for Use (EN.)	2
Mode d'emploi (FR.)	4
Gebrauchsanweisung (DE.)	6
Istruzioni per l'uso (IT.)	8
Gebruiksaanwijzing (NL.)	10
Bruksanvisning (SE.)	12
Instrukcja Używania (PL.)	14
Instruções de utilização (ES.)	16
用説明書 (JP.)	18
Vejledning til brug (DAN.)	20
Instruções de utilização (PT.)	22
Kullanım Talimatı (TR)	24
Uputstva za upotrebu (HR.)	26
Symbols	28

SMART BULB

Instructions for Use

EN

WS-IFU-037 Smart Bulb V01

PAGE 02

INDICATIONS

For use during urological and/or urological endoscopic procedures that require irrigation. Only sterile irrigation fluid should be used during urological endoscopic procedures.

CONTRAINDICATIONS

None known

USERS

Physicians and other qualified medical personnel.

PATIENT TARGET GROUP

Adult patients requiring diagnostic and/or therapeutic endoscopy in the upper urinary tract.

PREPARATION FOR USE

Inspect the package for any mechanical damage.

Do not use if package is damaged

CAUTION

All necessary manipulations during preparation and during intervention must be carried out in sterile field.

CAUTION

The Smart Bulb Irrigation System for Ureterorenoscopy should only be assembled and used by physicians and other qualified medical personnel, who are familiar with equipment used during endoscopic procedures.

POTENTIAL COMPLICATIONS

Any irrigation product may cause the following adverse effects:

- Excessive distension of urinary tract and hydronephrosis
- Extravasation of irrigating fluid and fornix rupture, flank pain
- Postoperative fever, urinary tract infection, sepsis and septic shock
- Transient haematuria, subcapsular haematoma

The patient should be routinely monitored in accordance with accepted standards for urological endoscopic procedures.

INTENDED USE AND DISPOSAL

The Smart Bulb Irrigation System for ureterorenoscopy is intended to be used on an individual patient, during a single urological procedure, and thereafter disposed of according to the hospital's medical waste handling procedure. It is not intended to be reprocessed and used again. If the device is reused, the patient would be at risk of infection and other complications.

DEVICE DESCRIPTION

Smart Bulb is a single use, sterile irrigation system that is used during endoscopy and in particular ureterorenoscopy, and helps to maintain a clear visual field by displacing blood, stone fragments, and cellular debris. Smart Bulb is designed for controlled irrigation through an endoscope or accessory during urological procedures.

INSTRUCTIONS FOR USE

1. Using the aseptic technique, open the sterile packaging and remove the unit.

2. Attach the Female Connector of Smart Bulb to the irrigation fluid supply.

4. Compress the Bulb to expel air. Then release, to fill the Bulb with irrigation fluid.


5. Connect the In-line Flow Switch to the scope.

SMART BULB EXTENSION

The Smart Bulb unit is supplied with an extension for the user's convenience. This can be inserted between the In-Line Flow Switch and an endoscope, if an assistant is required to operate the unit.

3. Ensure that the Switch is in the open position.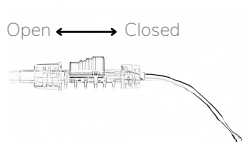

6. Compress the Bulb to deliver the desired volume of irrigation fluid. Release the Bulb to refill the unit.

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AB Yetkili Temsilcisi